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## PREFACE

Thank you for the confidence shown in Kid-Man. Every product is factory checked and left us in faultless condition. We wish you much success and we hope Kid-Man wheelchair will make a positive impact to your daily living.

## INDICATIONS

Intended to be used for persons having temporary or permanent mobility disorders. Suitable to be used both inside and outside.

## GENERAL NOTES

- The wheelchairs LightMan Comfort and LightMan Comfort Plus are delivered in a cardboard packaging and are ready for use. After unpacking please keep the cardboard packaging if possible. You may need it later for storage or for the return of the wheelchair. Once parcel is received, please check whether it is not damaged during transportation. If you see signs of damage, please make note to transport company on the same time. Please check content of package and if any of below parts are missing, get in touch with your local dealer.
- LightMan wheelchair is available in two different modifications - LightMan Comfort and LightMan Comfort Plus. "Plus" version is upgraded with drum brakes for assistant control. (Pic. No. 12) You may identify type of your wheelchair reading a label stucked on the armrest plate and metal cross brace underneath the seat. Round sticker with number (for example 41) on skirtguard indicates that size (seat width) of your chair is 41 cm.

Pic. No. 12

## SYMBOLS USED

Attention! This sign is used for very important parts concerning your safety. Please look at the operating instructions.

## CONTENT OF PACKAGE

- 1 x folded wheelchair
- 1 x pair of legrests
- 1 x pump (if rear wheels are inflatable)
- 1 x user manual
- 1 x set of adjustment tools

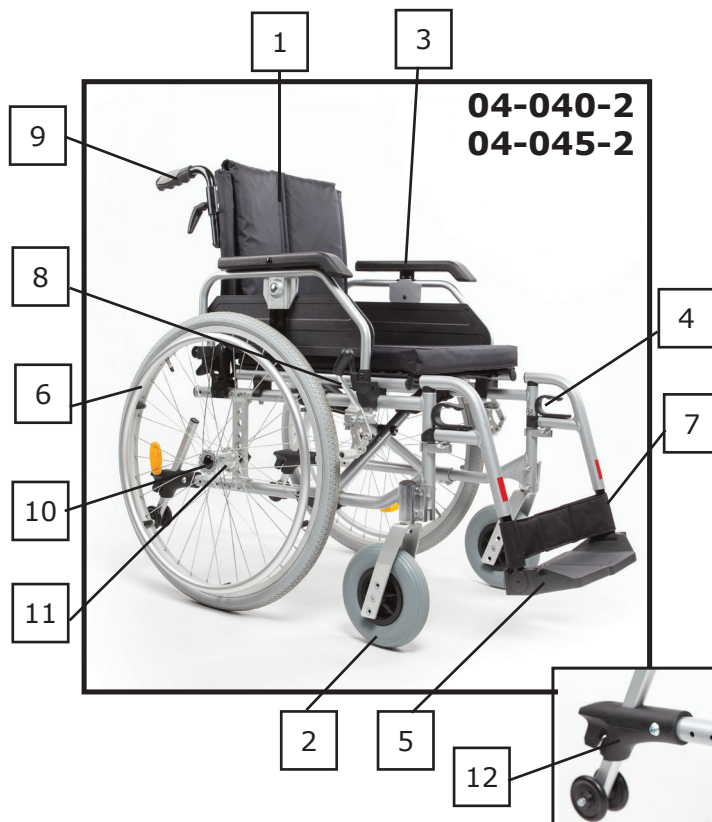
- Name of product
- Model
- Size sticker (seat width)
- Maximum load
- Manufacture
- Contact details
- Serial number

- Name of product
- Model
- Size sticker (seat width)
- Maximum load
- Manufacture
- Contact details
- Serial number

## PARTS OF THE PRODUCT

1. Backrest
2. Front castor
3. Armrest
4. Legrest swing lever
5. Legrest
6. Rear self propel wheel
7. Heel strap
8. Brake lever
9. Attendant push handles
10. Adapter block
11. Quick release axle
12. Anti-tipper

Warning! Completion of your wheelchair can differ a little from a wheelchair described in this manual due to continuous development of the product.

## UNFOLDING AND FOLDING THE WHEELCHAIR

Pic. No. 13

To unfold the wheelchair, push the cross brace apart-down direction until the seat pipes are completely set in the plastic holders. The seat cushion must be completely spread out. To fold wheelchair, first remove legrests or flip up footplates. After, place your hands on the seat cushion at the front and back and pull it up. In order to prepare wheelchair for transportation into car, we suggest to take off the rear wheels gently over the full floating axle. Danger of trapping your finger! Be careful not to grasp the seat pipe.

## **OPERATION**

For proper and safe use, we recommend you to follow safety instructions while using your wheelchair. This wheelchair was designed to ensure stability while seated. Every movement on the wheelchair affects the point of gravity, which may cause the wheelchair to tip over. To increase your safety, you may use seatbelt.

### **Reaching an object from the wheelchair - sitting**

Only the arms may be extended over the wheelchairs seat. The body and head should stay inside the limits of the seat.

### **Lateral transfer**

Transfer can be performed alone, assuming you are strong enough. Firstly, set the wheelchair as close to the place, which you want to sit on, as possible, whereby the steering wheels point forward. Activate the fixing brake and move your body towards the desired place. If there is no support beneath you while moving, please use a board if possible. Fold the legrest up and put them aside to make room. Bend your body forward, while leaning on the two armrests, and stand up. Then move your body toward the place, where you want to sit, while distributing your weight to your arms and hands.

### **Forward transfer**

If you are capable of getting up and standing on your own, you may change seats by leaning forward. If you have trouble with this movement, we advise you to seek assistance from an accompanying person to ensure an easy and safe transfer. Firstly, engage both brakes and make sure both steering wheels point towards the front.

Please bear in mind that you must not support yourself on the brakes. Grab the handle of the armrest firmly and support yourself on your arms, to stand up easily (before, make sure armrest are securely locked in place). The assistant may help you complete the transfer. Position yourself as close to the desired place as possible. When changing your position, please sit as far back as possible, to prevent screws from breaking, seat cushions from getting damaged, or the wheelchair from tipping over. Activate the two brakes; these brakes should in no way be used as support for the transfer. You must never support yourself on the footrests, while sitting or getting up.

## **Moving onto the sidewalk**

We recommend to have assistant to move onto sidewalk. Firstly, accompanying person positions the wheelchair straight in front of the curbside. After, tips the wheelchair backwards, until the steering wheels reach the curb. Lastly, pushes the wheelchair straight forward, until the back wheels hit the curb and when pushes the wheelchair again, until the back wheels go over the curbside.

## **Stairs**

We suggest seeking help from two people, one in front of the wheelchair and one behind it, due to the fact that hurdling stairs is more difficult and dangerous. The person behind the wheelchair, tips the wheelchair backwards until it is balanced. Then, pushes the wheelchair against the first step, grabbing the handle firmly, to prepare to lift the wheelchair. The person in front of the wheelchair grabs the sides of the frame in the front of the wheelchair and lifts the chair over the steps, while the other accompanying person steps on the next step, repeating the first process. Do not lower the wheelchair until you hurdled the last step of the person in front of the wheelchair. Carrying a wheelchair down the stairs is the same process as explained previously, in reversed order.

## **Ascending a ramp**

Lean your torso forward and use the hand rims with fast strokes, to keep up the proper speed. If you need a brake or relax on your journey, you must make sure to activate the two fixing brakes at the same time.

## **Descending a ramp**

Lean your torso backwards and let the hand rims go (flow through your hands), making sure you have the speed and direction under control. Avoid sudden directional changes, never try to ascend or descend a ramp diagonally and never try to turn the wheelchair while ascending. The wheelchair is not applicable as a seat for cars.

## SAFETY WARNINGS

Use slow speeds on gradients. Do not exceed the maximum gradient stated. Do not use escalators. Do not reverse down a gradient.

The wheelchair is only suitable for single occupancy.

Keep your feet on the footplates when moving. Do not stand on the footplates. Maintain proper balance at all times. Users should not move their centre of gravity out of the seating area.

Be aware of hazards in your environment, such as narrow doorways, steps, household appliances, children's toys, etc. Do not reach for items further than your arm will extend.

We recommend you consult your healthcare professional for advice about transferring to and from the wheelchair. The parking brakes should always be applied when transferring. The wheelchair brakes and parking brakes are not suitable for slowing the wheelchair down.

Warning! Do not attempt to lift the wheelchair by any removable parts. Lifting by removable parts of the wheelchair may result in injury to the user or damage to the wheelchair.

Warning! Do not attempt to move up or down an incline with a water, ice or oil film.

Warning! Do not attempt to ride over curbs or obstacles. Doing so may cause your wheelchair to tip over and cause bodily harm to you or damage to the wheelchair.

Warning! Never leave an unoccupied wheelchair on an incline or other place which may cause injury to surrounded people or things.

Warning! Do not attempt to stop the wheelchair while on a sloped surface.

Warning! All service, cleaning and adjustment should be performed while the wheelchair is unoccupied.

Warning! It is forbidden to exceed the maximum safe load of wheelchair.

The wheelchair must not be permanently exposed to temperatures greater than +40°C. Otherwise the frame as well as seat, backrest and armrests can heat up, which might cause burns.

## ARMREST OPERATION

Pic. No. 14

Pic. No. 15

Wheelchair is equipped with flip up and detachable armrest. To flip it up, please push plastic lever located in the front of armrest (Pic. No. 14). Keep it pushed and lift armrest gently. We recommend to flip armrest up getting in or out of wheelchair from side. While operating wheelchair, armrests should be locked. In case the need to completely remove armrest occurs, firstly flip it up as explained before. Once armrest is flipped up, find plastic handle in the rear of armrest (Pic. No. 15) and pull it out. Once pulled out, armrest is unlocked and you may take it out. To put it back, please repeat actions in opposite direction.

Pic. No. 16

### **Adjustment the height of the armrest**

You can easily adjust the height of the armrest. To do so, pull the height adjustment button on the side part and pull the armrest up or down. When you release the lever, the armrest locks in place automatically. (Pic. No. 16)

### **Adjustment the length of the armrest**

You can shift the armrests further forwards or backwards. To do so, simply push button located at the side of the armrest and pull this towards or backwards. When you release the button, the armrest locks in place automatically.

## REAR WHEEL OPERATION

Your chair has quick release axle, you need no tool to take off the rear wheel. Simply hold frame with one hand and push quick release axle located in the centre of rear wheel. (Pic. No. 17) Keep it pushed and pull rear wheel out. To put it back, repeat action in opposite way.

Pic. No. 17

## ANTI-TIPPER

Pic. No. 18

Both LightMan Comfort and LightMan Comfort Plus chairs are equipped with anti-tip wheels. They are factory in built, thus requires small adjustments only before use. The aim of anti tippers is to prevent the wheelchair from tipping backwards. They can also be used to help tilt the chair by an accompanying person. The height of the wheels can be adjusted. To do this, pull the silver-coloured disks towards you and slide the tube upwards or downwards along the bracket. If you want to take wheels off, unscrew bolt, take it out and pull device towards.

## SEAT HEIGHT ADJUSTMENT

Pic. No. 19

Thanks to front and rear wheels, you are able to adjust seat height. In order to adjust height of steering caster, untighten screw with allen key, pull it out, set caster on desired height and tighten again. (Pic. No. 19) In order to adjust height of rear wheel, take off the drive wheel to make it easier. To adjust the height of the drive wheel and, consequently, to raise or lower the height of the whole wheelchair, you first have to remove the drive wheel brackets. You can loosen the fastening nuts of the axle bracket with a spanner. Now you can remove the bracket and fasten it at the desired height. Screw the nut tight again and tighten the drive wheel. For safety reasons, we recommend to keep front and rear wheels in the same height.

### Seat depth adjustment

The depth of the seat is adjustable to ensure user's comfort and safety. To do so, firstly untightened two screws holding backrest and seat it in needed position. After it is set in place, secure screws.

### Wheelbase adjustment

The stability of the wheelchair is improved by lengthening the wheel base. On the other hand, shortening the wheel base improves manoeuvrability. It is recommended only for experienced wheelchair users. To set the wheelbase, please take off the rear wheel. After, loosen the four fastening screws that secure the wheel attachment. Place the attachment in needed position and secure again. Before use, ensure that all four screws are securely tightened. Make sure both rear wheels are set in the same position.

## BACKREST

Pic. No. 20

### **Backrest angle adjustment**

To adjust backrest angle, untighten the screw in the backrest from both sides and set in needed angle. Once in place, please tighten screw again.

### **Backrest height adjustment**

To adjust the backrest height, firstly take off the side parts and the back padding. Then loosen the handle on both sides and adjust the frame to the desired height. After, tighten handle again firmly.

### **Height adjustment of push handles**

To adjust the height of push handles loosen the knob on both sides and adjust the push handles in desired height. After, tighten knobs again firmly. We recommend to set push handles in the same height for comfortable use.

### **Tension straps (LightMan Comfor Plus only)**

You may adjust seating comfort using tension straps. Firstly, remove backrest padding. After, adjust level of seating comfort using tension straps. After, set backrest padding back.

## **BRAKES**

Your wheelchair has brakes mounted onto frame from both sides. Once delivered, brakes are adjusted properly to ensure sufficient braking. In order to apply brakes, please hold braking lever and push it forward. We recommend always use both brakes at the same time to ensure maximum safety. Due to normal wear and tear of rear wheel, braking efficiency may be reduced. In order to have sufficient braking, you must adjust position of brakes. To do so, please use allen key and untighten two screws located in bottom part of brakes. Then, push it forwards or backwards and tighten screws.

## **DRUM BRAKE**

LightMan Comfort Plus version is equipped with a drum brake, which should only be operated by an accompanying person via the brake lever located on the push handles. The drum brake should be used as both a locking brake and also as a service brake. When used as a service brake it provides well-regulated deceleration. The locking brake function is used for parking. To lock the brake, pull the brake lever and push the locking lever upwards. To release the locking brake, pull the brake lever upwards softly and the locking mechanism releases automatically. To achieve optimum braking efficiency, the braking power is set using the adjusting screw. The braking power is increased by unscrewing the adjusting screw and reduced by tightening it.

Pic. No. 21

Pic. No. 22

## LEGREST OPERATION

Pic. No. 23

Once delivered, legrests are packed in separate small cardboard. In order to install them, please take them out of package. The footrests must be installed in the front of chair. Firstly, please insert plastic part into metal tube and push it down. After, move legrest inside until you hear snap. Once locked in position, legrest stand firmly. (Pic. No. 23)

Do not mix left and right legrest! After installation, plastic footplate must be faced inwards.

Once leaving wheelchair, legrest must be flipped away or completely removed. Pull plastic lever upwards and move legrest outside.

Length of legrest can be adjusted to meet user's needs. Firstly, unscrew bolt using allen key, pull out bolt and tighten it again in needed position.

The angle of the foot plates can be adjusted. Firstly, unscrew the Allen bolt and adjust the foot plate to the required position by rotating the toothed ring. After, retighten the Allen bolt.

## TRANSPORTATION

To ensure safety and prevent damage while on board, all removable parts (armrests, legrests, rear wheels) must be removed.

## **CARE AND MAINTENANCE**

- Before each use of the wheelchair, the brakes should be checked.
- The wheelchair should be stored in a dry environment, away from direct sunlight. When in storage the wheelchair can be folded up. The wheelchair should be kept clean and dust free. This can be done with a duster or damp cloth and mild detergent.
- The user/professional assistant should routinely check the chair. It is obligated to replace teared parts (seat and back upholstery, tires).

## **SPARE PARTS**

Only original spare parts can be used for replacement. Please provide model of wheelchair and serial number to dealer or contact manufacturer directly. Manufacturer and dealer is not responsible for any damage occurred to product or user if non original spare parts are used and (or) repair was done in unauthorized facility.

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• We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB „Kasko Group“ after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB „Kasko Group“, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

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