LIGHTMAN START LIGHTMAN START PLUS

LENGVO LYDINIO NEIGALIOJO VEŽIMĖLIS
LIGHTWEIGHT WHEELCHAIR
VIEGLAIS INVALĪDU KRĒSLS
ОБЛЕГЧЁННОЕ ИНВАЛИДНОЕ КРЕСЛО

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- To unfold the wheelchair, push with your hands the cross brace apart-down direction until the seat pipes are completely set in the plastic holders. The seat cushion must be completely spread out. To fold wheelchair, first remove legrest or flip up footplates. After, place your hands on the seat cushion at the front and back and pull it up. In order to prepare wheelchair for transportation into car, we suggest to take off the rear wheels gently over the full floating axle. Danger of trapping your finger! Be careful not to grasp the seat pipe.

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OPERATION

- For proper and safe use, we recommend you to follow safety instructions while using your wheelchair. This wheelchair was designed to ensure stability while seated. Every movement on the wheelchair affects the point of gravity, which may cause the wheelchair to tip over. To increase your safety, you may use seatbelt.

Reaching an object from the wheelchair - sitting

Only the arms may be extended over the wheelchairs seat. The body and head should stay inside the limits of the seat.

Lateral transfer

Transfer can be performed alone, assuming you are strong enough. Firstly, set the wheelchair as close to the place, which you want to sit on, as possible, whereby the steering wheels point forward. Activate the fixing brake and move your body towards the desired place. If there is no support beneath you while moving, please use a board if possible. Fold the legrest up and put them aside to make room. Bend your body forward, while leaning on the two armrests, and stand up. Then move your body toward the place, where you want to sit, while distributing your weight to your arms and hands.

Forward transfer

If you are capable of getting up and standing on your own, you may change seats by leaning forward. If you have trouble with this movement, we advise you to seek assistance from an accompanying person to ensure an easy and safe transfer. Firstly, engage both brakes and make sure both steering wheels point towards the front.

Please bear in mind that you must not support yourself on the brakes. Grab the handle of the armrest firmly and support yourself on your arms, to stand up easily (before, make sure armrests are securely locked in place). The assistant may help you complete the transfer. Position yourself as close to the desired place as possible. When changing your position, please sit as far back as possible, to prevent screws from breaking, seat cushions from getting damaged, or the wheelchair from tipping over. Activate the two brakes; these brakes should in no way be used as support for the transfer. You must never support yourself on the footrests, while sitting or getting up.

Moving onto the sidewalk

We recommend to have assistant to move onto sidewalk. Firstly, accompanying person positions the wheelchair straight in front of the curbside. After, tips the wheelchair backwards, until the steering wheels reach the curb. Lastly, pushes the wheelchair straight forward, until the back wheels hit the curb and when pushes the wheelchair again, until the back wheels go over the curbside.

Stairs

We suggest seeking help from two people, one in front of the wheelchair and one behind it, due to the fact that hurdling stairs is more difficult and dangerous. The person behind the wheelchair, tips the wheelchair backwards until it is balanced. Then, pushes the wheelchair against the first step, grabbing the handle firmly, to prepare to lift the wheelchair. The person in front of the wheelchair grabs the sides of the frame in the front of the wheelchair and lifts the chair over the steps, while the other accompanying person steps on the next step, repeating the first process. Do not lower the wheelchair until you hurdled the last step of the person in front of the wheelchair. Carrying a wheelchair down the stairs is the same process as explained previously, in reversed order.

Ascending a ramp

Lean your torso forward and use the hand rims with fast strokes, to keep up the proper speed. If you need a brake or relax on your journey, you must make sure to activate the two fixing brakes at the same time.

Descending a ramp

Lean your torso backwards and let the hand rims go (flow through your hands), making sure you have the speed and direction under control. Avoid sudden directional changes, never try to ascend or descend a ramp diagonally and never try to turn the wheelchair while ascending. The wheelchair is not applicable as a seat for cars.

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- Wheelchair is equipped with flip up armrest. (Pic. No. 3) To flip it up, please push plastic lever located in the front of armrest. Keep it pushed and lift armrest gently. We recomend to flip armrest up getting in or out of wheelchair from side. While operating wheelchair, armrests should be locked. (Pic. No. 4)

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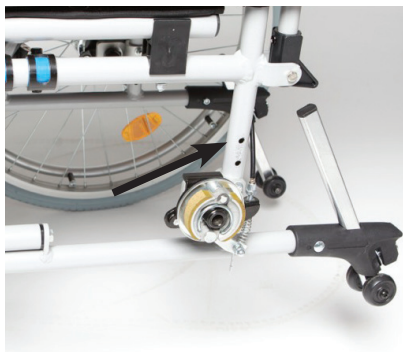
- Thanks to front and rear wheels, you are able to adjust seat height. In order to adjust height of steering caster, untighten screw with allen key, pull it out, set caster on desired height and tighten again. (Pic. No. 5) In order to adjust height of rear wheel, untighten 2 screws using spanner and set rear wheel adapter in needed height. (Pic. No. 5) After set in place, tighten it again. Please note, that rear wheel must be removed while making adjustment of adapter. For safety reasons, we recommend to keep front and rear wheels in the same height.

The wheelbase can be extended by turning the adapter by 180°. This increases the stability against tipping backwards.

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- Your chair has quick release axle, you need no tool to take off the rear wheel. Simply hold frame with one hand and push quick release axle located in the centre of rear wheel. (Pic. No. 7) Keep it pushed and pull rear wheel out. To put it back, repeat action in apposite way.

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- Both LightMan Start and LightMan Start Plus chairs are equipped with anti-tip wheels. They are factory in built, thus requires small adjustments only before use. The aim of anti tippers is to prevent the wheelchair from tipping backwards. They can also be used to help tilt the chair by an accompanying person. The height of the wheels can be adjusted. To do this, pull the silver-coloured disks towards you and slide the tube upwards or downwards along the bracket. If you want to take wheels off, unscrew bolt, take it out and pull device towards.

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• Your wheelchair has brakes mounted onto frame from both sides. Once delivered, brakes are adjusted properly to ensure sufficient braking. In order to apply brakes, please hold braking lever and push it forward. We recommend always use both brakes at the same time to ensure maximum safety. Due to normal wear and tear of rear wheel, braking efficiency may be reduced. In order to have sufficient braking, you must adjust position of brakes. To do so, please use allen key and untighten two screws located in bottom part of brakes. Then, push it forwards or backwards and tighten screws.

In case you move rear wheel adapter by 180° to extend rear wheel base, it is necessary to re-locate brakes. Please use allen key to untighten brake device, push it backwards and tighten again.

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- LightMan Start Plus version is equipped with a drum brake, which should only be operated by an accompanying person via the brake lever located on the push handles. The drum brake should be used as both a locking brake and also as a service brake. When used as a service brake it provides well-regulated deceleration. The locking brake function is used for parking. To lock the brake, pull the brake lever and push the locking lever upwards. To release the locking brake, pull the brake lever upwards softly and the locking mechanism releases automatically. To achieve optimum braking efficiency, the braking power is set using the adjusting screw. The braking power is increased by unscrewing the adjusting screw and reduced by tightening it.

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- Once delivered, legrests are packed in separate small cardboard. In order to install them, please take them out of package. The footrests must be installed in the front of chair. Firstly, please insert plastic part into metal tube and push it down. After, move legrest inside until you hear snap. Once locked in position, legrest stand firmly. (Pic. No. 11)

Do not mix left and right legrest! After installation, plastic footplate must be faced inwards.

Once leaving wheelchair, legrest must be flipped away or completely removed. Pull plastic lever upwards and move legrest outside.

Length of legrest can be adjusted to meet user's needs. Firstly, unscrew bolt using allen key, pull out bolt and tighten it again in needed position.

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• We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB „Kasko Group“ after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB „Kasko Group“, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

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