## STEELMAN XL

---

**NEĮGALIOJO VEŽIMĖLIS SUNKIASVORIUI  
HEAVY DUTY WHEELCHAIR  
PASTIPRINATI INVALIDŪ RATINĖI  
ИНВАЛИДНОЕ КРЕСЛО ДЛЯ ТЯЖЕЛОВЕСА**

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- To unfold the wheelchair, push with your hands the cross brace apart-down direction until the seat pipes are completely set in the plastic holders. The seat cushion must be completely spread out. To fold wheelchair, first remove legrest or flip up footplates. After, place your hands on the seat cushion at the front and back and pull it up. In order to prepare wheelchair for transportation into car, we suggest to take off the rear wheels using tools. Danger of trapping your finger! Be careful not to grasp the seat pipe.

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## **OPERATION**

- For proper and safe use, we recommend you to follow safety instructions while using your wheelchair. This wheelchair was designed to ensure stability while seated. Every movement on the wheelchair affects the point of gravity, which may cause the wheelchair to tip over. To increase your safety, you may use seatbelt.

### **Reaching an object from the wheelchair - sitting**

Only the arms may be extended over the wheelchairs seat. The body and head should stay inside the limits of the seat.

### **Lateral transfer**

Transfer can be performed alone, assuming you are strong enough. Firstly, set the wheelchair as close to the place, which you want to sit on, as possible, whereby the steering wheels point forward. Activate the fixing brake and move your body towards the desired place. If there is no support beneath you while moving, please use a board if possible. Fold the legrest up and put them aside to make room. Bend your body forward, while leaning on the two armrests, and stand up. Then move your body toward the place, where you want to sit, while distributing your weight to your arms and hands.

### **Forward transfer**

If you are capable of getting up and standing on your own, you may change seats by leaning forward. If you have trouble with this movement, we advise you to seek assistance from an accompanying person to ensure an easy and safe transfer. Firstly, engage both brakes and make sure both steering wheels point towards the front.

Please bear in mind that you must not support yourself on the brakes. Grab the handle of the armrest firmly and support yourself on your arms, to stand up easily (before, make sure armrest are securely locked in place). The assistant may help you complete the transfer. Position yourself as close to the desired place as possible. When changing your position, please sit as far back as possible, to prevent screws from breaking, seat cushions from getting damaged, or the wheelchair from tipping over. Activate the two brakes; these brakes should in no way be used as support for the transfer. You must never support yourself on the footrests, while sitting or getting up.

## **Moving onto the sidewalk**

We recommend to have assistant to move onto sidewalk. Firstly, accompanying person positions the wheelchair straight in front of the curbside. After, tips the wheelchair backwards, until the steering wheels reach the curb. Lastly, pushes the wheelchair straight forward, until the back wheels hit the curb and when pushes the wheelchair again, until the back wheels go over the curbside.

## **Stairs**

We suggest seeking help from two people, one in front of the wheelchair and one behind it, due to the fact that hurdling stairs is more difficult and dangerous. The person behind the wheelchair, tips the wheelchair backwards until it is balanced. Then, pushes the wheelchair against the first step, grabbing the handle firmly, to prepare to lift the wheelchair. The person in front of the wheelchair grabs the sides of the frame in the front of the wheelchair and lifts the chair over the steps, while the other accompanying person steps on the next step, repeating the first process. Do not lower the wheelchair until you hurdled the last step of the person in front of the wheelchair. Carrying a wheelchair down the stairs is the same process as explained previously, in reversed order.

## **Ascending a ramp**

Lean your torso forward and use the hand rims with fast strokes, to keep up the proper speed. If you need a brake or relax on your journey, you must make sure to activate the two fixing brakes at the same time.

## **Descending a ramp**

Lean your torso backwards and let the hand rims go (flow through your hands), making sure you have the speed and direction under control. Avoid sudden directional changes, never try to ascend or descend a ramp diagonally and never try to turn the wheelchair while ascending. The wheelchair is not applicable as a seat for cars.

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• Wheelchair is equipped with removable armrest. To remove it, please push a metal pin located in the front of armrest. (Pic. No. 2) Keep it pushed and lift armrest gently. We recommend to remove armrest getting in or out of wheelchair from side. While operating wheelchair, armrests should be locked.

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- Your wheelchair has brakes mounted onto frame from both sides. Once delivered, brakes are adjusted properly to ensure sufficient braking. In order to apply brakes, please hold braking lever and push it forward. We recommend always use both brakes at the same time to ensure maximum safety. Due to normal wear and tear of rear wheel, braking efficiency may be reduced. In order to have sufficient braking, you must adjust position of brakes. To do so, please use allen key and untighten a screw located in the top part of brakes. Then, push it to the front or back and tighten a screw.

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- Once delivered, legrests are packed in a separate small cardboard. In order to install them, please take them out of package. The footrests must be installed in the front of chair. To add the legrest, place the legrest on the two posts located on the frame. Then swing the legrest inwards until it locks in position. (Pic. No. 8)  
Do not mix left and right legrest! After installation, plastic footplate must be faced inwards.

Once leaving wheelchair, legrest must be flipped away or completely removed. Pull metal lever upwards and move legrest outside. Length of legrest can be adjusted to meet user's needs. legrest. In order to adjust length of legrest, simply push metal button and pull the inned tube in or out depending you want to increase or decrease the length.

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• We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB „Kasko Group“ after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB „Kasko Group“, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

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